# STABILO MULTISEAT

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# car safety seat

with the padding adjustable to a child's body

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### INDICATION

The seat is designed for children with low muscle tension or with CP and spasticity.

### ACTION

Thanks to the use of STABILO system you can adjust both the shape and the hardness of the seat according to the needs of each person.

- correct an improper position of the pelvis
- stabilize the trunk with side supports thus prevent it from deformities
- shape appropriate, asymmetrical support for the back in case of kyphosis, scoliosis or hyperlordosis
- thanks to the adjustable headrest you will correctly protect the head and cervical spine (neck)

### USE

- as a child car safety seat (certified according to ECE R44/04)
- as an orthopaedic seat stabilizing both the back and head and being mounted to a chair/armchair
- as an indoor/outdoor swing
- as a short distance litter – you can use the swing straps when two persons carry a child
- as an insert to the rocking chair

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SAFETY

The MULTISEAT is a child car safety seat certified according to the European safety standards ECE R44/04. Both the mounting straps it is equipped with and the necessity of strapping through and fastening the car safety belt provide holding of the device in the event of a sudden deceleration.

COMFORT

The STABILO MULTISEAT is the only child car safety seat designed specifically for special needs children. It is based on the STABILO system of vacuum posture cushions, which can be freely shaped to meet a child's needs. This way even a long ride is comfortable.

STABILIZATION

The STABILO cushions being a padding of the MULTISEAT serve primarily to position the body in a proper way – from the pelvis to the head. Consequently, a child does not experience further deformities even during a ride.

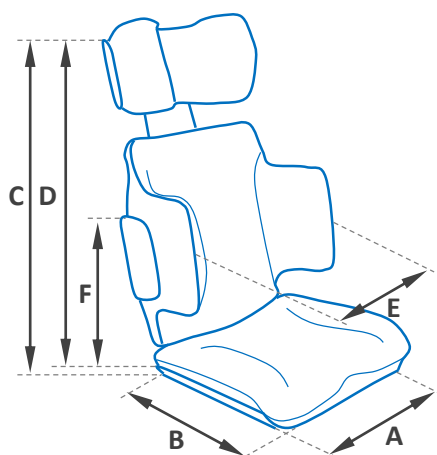
FLEXIBILITY

We can adjust the headrest in the STABILO MULTISEAT, as well as the width of the side supports. There is a possibility to attach a wedge-shaped metal platform to gain a tilt.

VERSATILITY

The STABILO MULTISEAT can be used at home, in a car and outdoor in various situations.

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### ADDITIONAL ACCESSORIES

- 1. 4-points holding vest
- 2. STABILO SHORTS for positioning of the pelvis and legs abduction
- 3. wedge-shaped platform for tilt

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